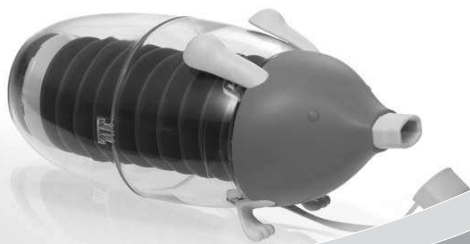
**INSTRUCTIONS FOR USE**

Read these Instructions for use before using the WATCHHALER for the first time

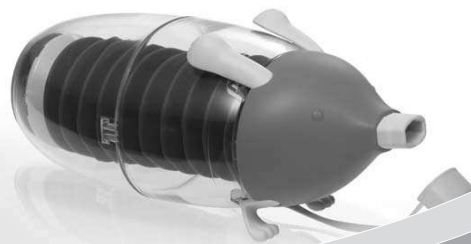
NDC# 52747-0515-01, Rx Only

This product is covered under following patents:  
US 6681762, US 6401710, US 6463929, US 6571791, US D569968, US D569500

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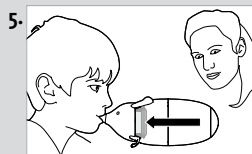
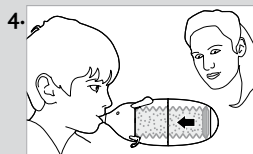
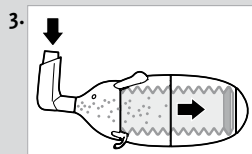
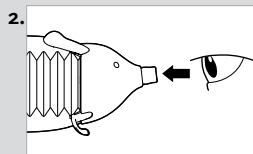
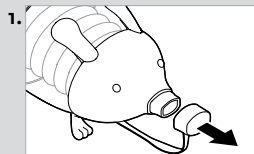
**BRIEF INSTRUCTION**

- 1 Remove the protective cover from the mouthpiece of the Watchhaler.
- 2 Ensure before each filling, that the folding balloon is empty.
- 3 Remove the cap of the MDI and place its mouthpiece onto the Watchhaler mouthpiece. Release the MDI.
- 4 Remove the MDI and give the Watchhaler to the patient. Watch closely to the movement of the balloon during inhalation.
- 5 At the end of a successful inhalation, the folding balloon is compressed.

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**1 INDICATIONS FOR USE**

The Watchhaler is an inhalation system (Spacer or Holding Chamber) for the inhalation of approved liquid compressed gas metered dose inhaler systems (MDI) for the therapy of the lower respiratory system. The Watchhaler can be applied to awake, actively cooperating children, starting from the age of 3 and older under supervision. The intended environments for use include the home, hospitals, and clinics.

**2 NOTES**

- Do not leave the Watchhaler unattended with children. This is a medical device, not a toy.
- This product contains no latex.
- MDI = Metered Dose Inhaler

**3 CAUTION**

- The Watchhaler is designed for single patient use only. A change of patient is not permissible, even after hygienic treatment.
- In order to avoid a health risk e.g. an infection due to contamination, it is necessary to follow the cleaning instructions in Chapter 8. Follow these instructions prior to first application.
- Product should be replaced if parts are damaged or missing.
- Administer only one puff at a time.
- If unsure on how to use this product, talk to your physician or pharmacist.
- Rx Only

**4 LIABILITY**

The Activaero GmbH and/or US Pharmaceutical Corporation are not responsible for damages or malfunctions, which occur due to inappropriate or non-intended activities of the operator.

**5 SCOPE OF DELIVERY**

Ensure that all following specified parts are available and undamaged, when unpacking the Watchhaler transport box. If you should have received an incomplete or damaged delivery, please contact your specialist dealer.

FIGURE 1: 1) Head piece cover with protective cover; 2) Head piece holder; 3) Folding balloon; 4) Slide ring; 5) Housing; 6) Instructions for use (without illustration)

**6 APPLICATION**

Please observe the cleaning instructions prior to first use. Examine the Watchhaler before each use for foreign bodies (danger of suffocation!) and damages. Ensure before treatment, that the Watchhaler works as intended. The folding balloon must fold itself up when inhaling and then return to its completely expanded shape. The Watchhaler can be used with most of the approved MDIs. It is, however, important, that the mouthpiece of the MDI fits onto the Watchhaler mouthpiece. The MDI can generally be applied to two different positions on the Watchhaler mouthpiece when filling:

The Watchhaler cannot be used, if the MDI:

- cannot be clearly positioned on the mouthpiece of the Watchhaler.
- cannot be manually initiated.

FIGURE 2: MDIs with large or round mouthpieces cover the complete yellow mouthpiece of the Watchhaler.

FIGURE 3: If the Watchhaler is used with a MDI with a smaller or oval mouthpiece, it will only enclose half of the yellow mouthpiece of the Watchhaler.

**6.1 FILLING THE WATCHHALER****IMPORTANT NOTES ON FILLING THE WATCHHALER:**

- Always ensure before filling, that the Watchhaler is not damaged and that no parts have broken off. The Watchhaler must be replaced immediately, if it is damaged.
- Observe the instructions for use of your MDI and/or follow the instructions of your physician or pharmacist with regard to dosage and application of your MDI with the Watchhaler.

FIGURE 4: Remove the protective cover from the mouthpiece of the Watchhaler.

FIGURE 5: Ensure before each filling, that the folding balloon is empty and fully expanded.

FIGURE 6: Remove the cap of the MDI and place its mouthpiece onto the Watchhaler mouthpiece. Release the MDI.

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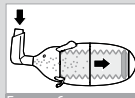
FIGURE 6  
ILUSTRACIÓN 6

## 6.2 INHALATION

If possible, train the use of the Watchhaler playfully and without medicine together with the child before application. If necessary, demonstrate how to inhale (without medicine), so that the folding of the balloon is visible. Pay attention to the following sequence during inhalation:

1. Breathe out completely
2. Put the mouthpiece (yellow part) into the mouth
3. Suck on the mouthpiece, as if it was a straw
4. Release the mouthpiece and exhale (Folding balloon expands again)

### IMPORTANT NOTES ON INHALATION WITH THE WATCHHALER:

- Give the child the already filled Watchhaler.
- The inhalation must take place immediately after filling.
- If necessary, help the child hold the Watchhaler correctly (like a drinking bottle).
- Pay attention to the movement of the slide ring during the inhalation (see Fig. 8).
- Do NOT exhale into the device.
- The inhalation has been completed after a deep breath, the child can then continue to breath freely.

FIGURE 7: Filled Watchhaler

FIGURE 8: Folding balloon is compressed during the inhalation.

## 7 DISASSEMBLING AND ASSEMBLING THE WATCHHALER

### 7.1 DISASSEMBLING THE WATCHHALER

FIGURE 9: Releasing the head piece from the housing: Press the small tabs on the Watchhaler forwards. The housing may not be disassembled further.

FIGURE 10: Remove the protective cover from the mouthpiece of the Watchhaler.

FIGURE 11: Press firmly onto the mouthpiece of the Watchhaler to separate the head piece holder from the head piece cover. The protective cover remains connected to the mouthpiece cover.

FIGURE 12: Remove the folding balloon, by holding the housing with the opening pointed downwards.

FIGURE 13: Remove the slide ring from the folding balloon.

### 7.2 ASSEMBLING THE WATCHHALER

Assemble the Watchhaler, as described in the illustrations. Ensure that the individual components are intact. Ensure furthermore, that the openings on the rear side of the Watchhaler are not clogged or damaged.

FIGURE 14: Attach the slide ring to the folding balloon as illustrated. Make sure that the ribs of the folding balloon are not dented. FIGURE 15: Hold the housing with the opening upwards and insert the folding balloon. Check the correct seat of the folding balloon.

FIGURE 16: Hook the ears of the head piece holder (yellow part) into the housing until the tabs engage. The Watchhaler may not be used any longer, if the fastening mechanics are defective. Make sure, that the folding balloon is well positioned between the head piece holder and the housing.

FIGURE 17: Put the head piece cover (orange part) onto the head piece holder. The connection must engage audibly. Close the mouthpiece of the Watchhaler with the protective cover.

## 8 CLEANING INSTRUCTIONS

Clean the Watchhaler components, i.e. the folding balloon, head piece holder and head piece cover with protective cover, prior to first use, then weekly. The housing and the slide ring must only be cleaned, if they are contaminated. Verification of the suitability of the Watchhaler for effective cleaning was provided by an independent test laboratory using the recommended and mentioned techniques. Access to the test documentation is possible on request.

### Cleaning Procedure:

- Disassemble the Watchhaler as described in chapter 7.1 - Do not remove the orange cap.
- Soak the Watchhaler components folding balloon, head piece holder and head piece cover with protective cover for 15 minutes in lukewarm water with liquid detergent. Agitate gently.
- Rinse the Watchhaler components in clean water.
- Shake out excess water.
- Let air dry in vertical position for at least 4 hours. Drying should not take place in rooms with high air humidity.
- Assemble the Watchhaler as described in chapter 7.2 when the unit is completely dry and ready for use.

## 9 STORAGE AND TRANSPORT

Place the protective cover on the mouthpiece for storage and transport. Do not put the Watchhaler into a plastic bag when it is still damp. Do not keep the Watchhaler in the car or in damp environments (e.g. in the bathroom). Do not allow acids and caustic solutions as well as solvents to come into contact with the Watchhaler components. The manufacturer and distributor are not liable, if these instructions are ignored.

## 10 SERVICE LIFE

For hygienic reasons the service life of the Watchhaler is one year and/or 1000 applications and must be replaced after this time. Replace the Watchhaler immediately, in case of visible damages.

## 11 DISPOSAL

The cleaned Watchhaler components can be disposed together with the domestic refuse. The packing can be disposed together with used paper.

## 12 TECHNICAL DATA

Maximum inhalation flow: .....915 cu. in./min (15 l/min)  
Complete inhalation volume: .....18.3 cu. in. (300 ml)  
Weight: .....6.0 oz (170 g)  
Dimensions (L x W x H): .....8.6 in x 3.6 in x 3.3 in (218mm x 92mm x 84mm)

### MATERIALS

Folding balloon, valve membrane: .....Silicone  
Head piece, protective cover, slide ring and valve cap: .....PP plastic  
Housing: .....ABS plastic

### TEMPERATURE RANGES

Storage and transport: .....-40 to 176 °F (-40 to 80 °C)  
Use: .....59 to 104 °F (15 to 40 °C)  
Preparation: .....Housing: up to 176 °F (80 °C)  
Other components: .....up to 248 °F (120 °C)

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